


# 1500 N95

**N95 HEALTHCARE  
PARTICULATE  
RESPIRATOR  
& SURGICAL MASK**

## FEATURES

- Meets CDC guidelines for protection against TB
- NIOSH certified to have a filter efficiency of 95% or greater against non-oil based particulates
- Dura-Mesh® shell resists collapsing in regular use and even under hot and humid conditions
- Contour molded nosebridge for extra comfort
- Latex Free
- Soft foam nose cushion for an easy and comfortable fit
- Softspun® inner lining is soft to the touch
- Fluid resistant
- Bacterial filter efficiency >99.9%
- Available in 5 color coded sizes
- Highest ASTM level of fluid resistance on the market (160 mm Hg)

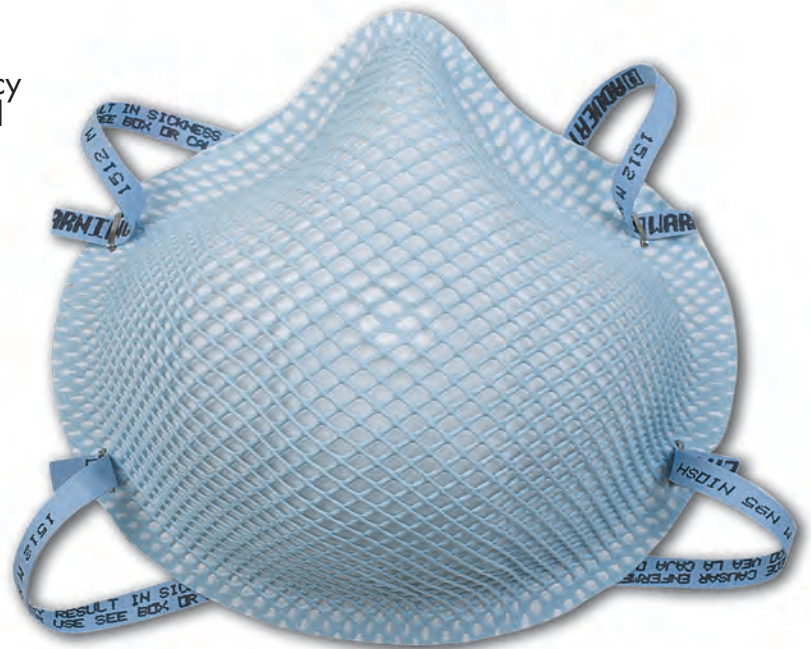

## BENEFITS

### **EASY FIT, MORE COMFORT AND DEPENDABLE**

The molded nose bridge with soft foam nose cushion provides the wearer an easy comfortable, dependable fit. There's no metal noseband to adjust and the seal is maintained without pressure points.

### **LAST LONGER, USE FEWER**

The Dura-Mesh® shell, exclusive to Moldex, prevents filter media contact by contaminated gloves or hands. The shell resists collapsing, even in heat and humidity. So the respirator holds its shape longer. The result is fewer respirators used and lower costs.

### **LATEX FREE RESPIRATOR AND STRAPS**

The non-latex straps don't irritate. They are color coded, don't require prestretching, and they are securely attached.



### **MOLDEX TECHNICAL SUPPORT**

Moldex provides technical assistance including help with fit testing, a Train-the-Trainer Program, a manual on How To Set-Up Your Respiratory Protection Program, plus a Toll-Free Technical Help line. Just call 800/421-0668 ext. 512/550 or 310/837-6500 ext. 512/550.

**MOLDEX**  
Ideas that wear well!

# 1500 N95

## N95 HEALTHCARE PARTICULATE RESPIRATOR & SURGICAL MASK

### WARNING TO USER

Non-latex straps are used in the headbands of these respirators. However, individuals highly sensitive to natural rubber latex may have an allergic reaction. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards. This product has not been sold with warning or use instructions for personnel involved in industrial or related situations. If you are considering such uses, first contact the Moldex Technical Services Department at 800/421-0668 or 310/837-6500 ext. 512/550.

### INTENDED USE

The various models of Moldex Type N95 Healthcare Particulate Respirators and Surgical Masks meets CDC Guidelines for TB Exposure Control within healthcare facilities. These devices are also intended to be worn by healthcare personnel during surgical procedures to protect both the patient and healthcare personnel from the transfer of microorganisms, body fluids and particulate material.

### CONTRAINDICATIONS

1. Not for use with beards, or other facial hair that prevents direct contact between the face and sealing surface of the respirator.
2. Eyewear must not prevent direct contact between the face and sealing surface of the respirator.
3. Not to be used on children.

### RESTRICTIONS

1. Before use, a written respiratory protection program in accordance with 29 CFR 1910.134 (f) must be implemented.
2. This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
3. Prior to each use, inspect filter media and strap attachment area for tears and damage. Staple or strap attachment perforations do not affect NIOSH approval.
4. If respirator is damaged, distorted, a proper fit cannot be obtained or breathing becomes difficult, leave contaminated area as soon as possible and replace respirator.
5. If the respirator comes in contact with blood or fluids, including body fluids, leave contaminated area as soon as possible and discard and replace the respirator.
6. Do not alter, modify or abuse this respirator.
7. Store unused respirators in box in a clean, dry, non-contaminated area.
8. Dispose of respirator according to your employer's policy and local regulations.
9. Do not reuse or store for reuse or hang around neck unless your employer specifically authorizes reuse.
10. When used for surgical procedures, discard after every use.
11. Use respirator before the "use by" expiration date printed on box.

### DESCRIPTION


Moldex N95 Healthcare Particulate Respirator is disposable and designed to provide respiratory protection for the wearer. This product has been tested and certified by NIOSH to have a filter efficiency level of 95% or greater against solid and liquid (non oil-based) aerosols.

- 1 Tested in accordance with NIOSH 42 CFR 84
- 2 Passed ASTM F 1862 © 160 mm Hg


### MOLDEX-METRIC, INC.

10111 W. Jefferson Blvd. Culver City, CA 90232  
 800/421-0668 or 310/837-6500 FAX 310/837-9563  
 www.moldex.com e-mail: sales@moldex.com  
 Moldex, Dura-Mesh and Softspun are registered trademarks.  
 The Mesh Design is a trademark of Moldex.  
 U.S. Patents #D285,374, #4,850,347, #4,384,577.  
 Made in the U.S.A. 9700-856 REV A 08/07

### NIOSH LABEL - 1510, 1511, 1512, 1513, 1517



INOVEL LLC  
 CULVER CITY, CALIFORNIA, U.S.A.  
 866-546-6835



THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC-	PROTECTION <sup>1</sup>	RESPIRATOR					CAUTIONS AND LIMITATIONS <sup>2</sup>
		1510	1511	1512	1513	1517	
84A-4333	N95	X					A B C J M N O P
84A-0013	N95		X	X	X	X	A B C J M N O P

1. Protection
 

N95-Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply
2. Cautions and Limitations
  - A- Not for use in atmospheres containing less than 19.5 percent oxygen.
  - B- Not for use in atmospheres immediately dangerous to life or health.
  - C- Do not exceed maximum use concentrations established by regulatory standards.
  - J- Failure to properly use and maintain this product could result in injury or death.
  - M- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
  - N- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
  - O- Refer to User's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
  - P- NIOSH does not evaluate respirators for use as surgical masks.

### TECHNICAL SERVICE HELP LINE

For additional information phone 800/421-0668 or 310/837-6500, ext. 512/550.

Part Number	Size	Strap Color	Packaging - Total, Masks/Box, Box/Case
1510	X-Small	Yellow	160 Masks/Case, 20 Masks/Box, 8 Boxes/Case
1511	Small	White	
1512	Medium	Blue	
1513	Large	Tan	
1517	Low Profile	Green	

### LIMITED WARRANTY IMPORTANT NOTICE TO PURCHASER

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. **Exclusive Remedies:** damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

