

FITTING INSTRUCTIONS FOR MOLDEX® DISPOSABLE RESPIRATORS

THESE ARE DISPOSABLE. LUNGS AREN'T.




Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand.



Place respirator under chin with molded nose contour (narrow end) up. Do not wear with only one strap. When changing from any model/size to another respirator, you must fit test. (2200N95 Flange must be uncreased inside respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to around neck.)



Pull shorter bottom strap over head, below ears, to around neck. Adjust respirator for comfortable fit.

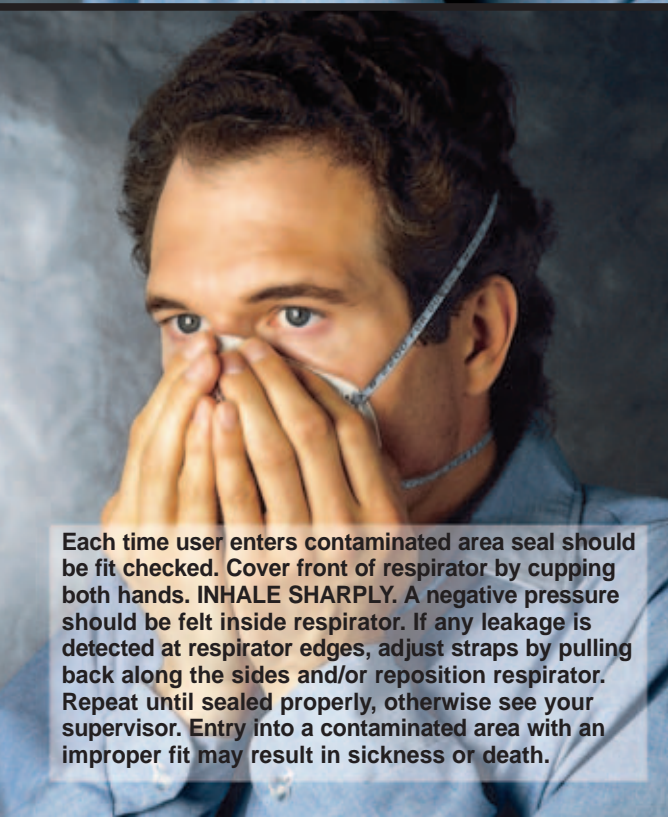

Each time user enters contaminated area seal should be fit checked. Cover front of respirator by cupping both hands. INHALE SHARPLY. A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.



WARNING

Failure to follow all instructions and warnings on the proper use of this respirator may reduce respirator effectiveness, wearer protection and may result in sickness and/or death. It is the user's responsibility to obtain a proper fit. For additional information refer to your supervisor and see box/bag or phone 800/421-0668 ext. 550 or 310/837-6500 ext. 550.

USER WARNING

Users must follow instructions each time respirator is worn. It's the wearer's responsibility to obtain a proper fit. To choose the correct size you must fit test and user seal check. If you can not obtain a proper fit do not enter the contaminated area. OSHA regulation 1910.134 (f) requires that the user be fit tested before being assigned a respirator. User must be clean-shaven. Any facial hair, such as beards or long sideburns, may prevent the respirator from fitting properly.

MOLDEX-METRIC, INC.

10111 W. Jefferson Blvd.
Culver City, CA 90232
800/421-0668 310/837-6500
FAX 310/837-9563

www.moldex.com

e-mail: sales@moldex.com
U.S. Patents #D285,374, #4,850,347,
#4,384,577, #5,149,468.

Moldex is a registered trademark.
Made in the U.S.A. 9700-712 REV E 8/05