

# 1500 SERIES

## N95 HEALTHCARE PARTICULATE RESPIRATORS & SURGICAL MASKS FITTING INSTRUCTIONS



# 1



Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstraps to fall below hand.

# 2



Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must not be creased inside respirator. Raise top strap to top back of head. Pull shorter bottom strap over head, below ears, to around neck. Do not wear with only one strap because it may affect fit.

# 3



Adjust respirator for comfortable fit.

# 4



Each time user enters contaminated area seal should be fit checked. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, adjust straps by pulling back along the sides and/or reposition respirator. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.

**MOLDEX**<sup>®</sup>  
Ideas that wear well.<sup>®</sup>



Printed on Recycled Paper

### ALERT

Failure to follow all instructions and warnings on the proper use of this respirator may reduce respirator effectiveness, wearer protection and may result in sickness and death. For additional information consult your supervisor, read all information on box, and phone +1 (800) 421-0668 ext. 512/550.

### MOLDEX-METRIC, INC.

10111 W. Jefferson Blvd., Culver City, CA 90232  
Tel: +1 (800) 421-0668 or +1 (310) 837-6500 Fax: +1 (310) 837-9563  
Email: sales@moldex.com www.moldex.com

Moldex Technical Service Department: +1 (800) 421-0668  
or +1 (310) 837-6500 ext. 512/550

U.S. Patents #D285,374 and #4,850,347.  
Moldex, Ideas that wear well, Softspun and Dura-Mesh are registered trademarks.  
The mesh design is a trademark of Moldex-Metric, Inc. Made in U.S.A.  
9700V-776 REV F 07/09

### USER INSTRUCTION

Users must follow instructions each time respirator is worn. It's the wearer's responsibility to obtain a proper fit. To choose the correct model/size Moldex Respirator you must fit test and user seal check. If you cannot obtain a proper fit do not enter the contaminated area. OSHA regulation 29 CFR 1910.134 (f) requires that the user must be fit tested before being assigned a respirator and annually thereafter. Use Moldex Qualitative Fit Test Kit. User must be clean-shaven. Any facial hair, such as beards or long sideburns, may prevent the respirator from fitting properly. Eyewear must not interfere with face to face-piece seal. Re-fit test when there are any changes to the wearers physical condition that may affect fit.

### WARNING

This product contains no components made from natural rubber latex. However, individuals highly sensitive to natural rubber latex may have an allergic reaction. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards. This product has not been sold with warnings or use instructions for personnel involved in industrial or related situations. If you are considering such uses, first contact the Moldex Technical Services Department at +1 (800) 421-0668 or +1 (310) 837-6500 ext. 512/550.

# 3200 SERIES

## N95 HEALTHCARE PARTICULATE RESPIRATORS & SURGICAL MASKS FITTING INSTRUCTIONS



# 1



Hold respirator in hand with molded nose contour (narrow end) at finger tips, allowing headstrap to fall below hand.

# 2



Place respirator under chin with molded nose contour (narrow end) up. Nose cushion must not be creased inside respirator. Pull head halo to top of head so it rests on the crown.

# 3



Adjust halo on crown of head to provide a comfortable and secure fit.

# 4



Each time before entering a contaminated area, perform a user seal check. Cover front of respirator by cupping both hands. **INHALE SHARPLY.** A negative pressure should be felt inside respirator. If any leakage is detected at respirator edges, reposition respirator and/or select another size. Repeat until sealed properly, otherwise see your supervisor. Entry into a contaminated area with an improper fit may result in sickness or death.

**MOLDEX**<sup>®</sup>  
Ideas that wear well.<sup>®</sup>



Printed on Recycled Paper

### ALERT

Failure to follow all instructions and warnings on the proper use of this respirator may reduce respirator effectiveness, wearer protection and may result in sickness and death. For additional information consult your supervisor, read all information on box, and phone +1 (800) 421-0668 ext. 512/550.

#### MOLDEX-METRIC, INC.

10111 W. Jefferson Blvd., Culver City, CA 90232  
Tel: +1 (800) 421-0668 or +1 (310) 837-6500 Fax: +1 (310) 837-9563  
Email: sales@moldex.com www.moldex.com

Moldex Technical Service Department: +1 (800) 421-0668  
or +1 (310) 837-6500 ext. 512/550

U.S. Patents #D285,374 and #4,850,347.  
Moldex, Ideas that wear well, Softspun and Dura-Mesh are registered trademarks.  
The mesh design is a trademark of Moldex-Metric, Inc. Made in U.S.A.  
9700V-776 REV F 07/09

### USER INSTRUCTION

Users must follow instructions each time respirator is worn. It's the wearer's responsibility to obtain a proper fit. To choose the correct model/size Moldex Respirator you must fit test and user seal check. If you cannot obtain a proper fit do not enter the contaminated area. OSHA regulation 29 CFR 1910.134 (f) requires that the user must be fit tested before being assigned a respirator and annually thereafter. Use Moldex Qualitative Fit Test Kit. User must be clean-shaven. Any facial hair, such as beards or long sideburns, may prevent the respirator from fitting properly. Eyewear must not interfere with face to face-piece seal. Re-fit test when there are any changes to the wearers physical condition that may affect fit.

### WARNING

This product contains no components made from natural rubber latex. However, individuals highly sensitive to natural rubber latex may have an allergic reaction. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids, including body fluids. OSHA has not set a permissible exposure level for airborne biohazards. This product has not been sold with warnings or use instructions for personnel involved in industrial or related situations. If you are considering such uses, first contact the Moldex Technical Services Department at +1 (800) 421-0668 or +1 (310) 837-6500 ext. 512/550.