


4300 SERIES

P95 PARTICULATE RESPIRATOR W/ SMARTSTRAP, AND FOAM FACE CUSHION

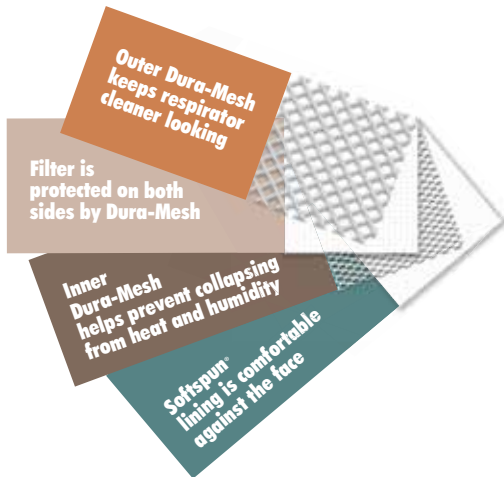

MORE AIR. MORE WEAR.

Easier breathing and more comfort are what you get when you wear an AirWave® respirator. The larger surface area of the patented wave design means more air flows in and out of the mask.


MORE COMFORT. MORE USE. MORE DURABILITY.

The unique FlexFit® design moves with your face for comfort all day long. The exclusive cool and comfortable SmartStrap allows you to adjust the mask for a custom fit. The SmartStrap also allows the respirator to hang around the neck when not in use. It has a soft foam face cushion for no pressure points. More comfort means more compliance. The Dura-Mesh® shell prevents collapsing from heat and humidity so your mask lasts longer. Longer use means less waste and lower costs. Plus it's also metal free.


Adjustable



Hangable

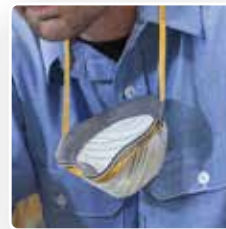

Foam Face Cushion

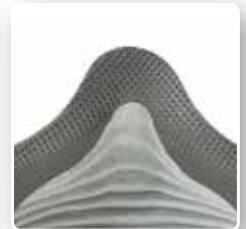

P95 PROTECTION

NIOSH approved 95% efficient against oil and non-oil based particulates.

FEATURES

- Exclusive easy-breathing wave design.
- 95% efficient against oil and non-oil based particulates.
- FlexFit design moves easily with your face and stays in place.
- Dura-Mesh shell resists collapsing in heat and humidity.
- Soft, cloth SmartStrap is both HANGABLE and ADJUSTABLE.
- Foam face cushion for added comfort and no pressure points.
- Meets heat and flame resistance in accordance with ANSI/ISEA 110-2009 Section 7.11.1.
- Metal-free.
- 100% PVC-Free for a greener alternative.

P95 APPLICATIONS:

Dusty environments such as welding, soldering and brazing, cutting and metal pouring, grinding, sanding, sweeping, bagging, stone quarrying, metal machining, cement, underground mining, polishing, buffing, lime, poultry and textiles, pharmaceutical manufacturing, petrochemical manufacturing, dry chemical handling, conventional and nuclear power generation, and paper processing.

4300 SERIES

P95 PARTICULATE RESPIRATOR W/ SMARTSTRAP, AND FOAM FACE CUSHION



Product Code	Size	Flange Type	Strap Material	Adjustable/Hangable SmartStrap	Display Quantity	Case Quantity
4300P95	M/L	Full	Cloth	Yes	8/Box	10 Boxes

WARNING TO USER

- Follow all instructions and warnings on the use of this respirator and wear during all times of exposure. Failure to do so will reduce respirator effectiveness, wearer protection and may result in sickness or death.
- Before use, the user must first be trained by the employer in proper respirator use, in accordance with applicable safety and health standards, for the contaminant and exposure level in the assigned work area.
- The particulates and other contaminants which can be dangerous to your health include those which you cannot see, taste, or smell.
- This product has not been sold with warnings or use instructions for personnel involved in healthcare or related situations, where there may be the possibility of contact with disease or biological hazards. If you are considering such uses, first call the Moldex Technical Service Department: +1 (800) 421-0668 or +1 (310) 837-6500 ext. 512/550

LIMITED WARRANTY IMPORTANT NOTICE TO PURCHASER

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. Exclusive Remedies: damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex, products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.


MOLDEX TECHNICAL ASSISTANCE

For more information call +1 (800) 421-0668 or +1 (310) 837-6500 ext. 512/550 or E-mail: sales@moldex.com, tech@moldex.com or visit www.moldex.com.




MOLDEX-METRIC, INC.
10111 Jefferson Blvd.
Culver City, CA 90232
TEL: +1 (800) 421-0668 or
+1 (310) 837-6500
FAX: +1 (310) 837-9563
E-mail: sales@moldex.com
www.moldex.com

CANADIAN CUSTOMER SERVICE
Tel: +1 (800) 421-0668, Ext. 550
Fax: +1 (310) 837-9563



Moldex-Metric, Inc.
CULVER CITY, CALIFORNIA, U.S.A.
+1 (800) 421-0668



THESE RESPIRATORS ARE APPROVED ONLY IN THE FOLLOWING CONFIGURATIONS:

TC-	PROTECTION ¹	RESPIRATOR	CAUTIONS AND LIMITATIONS ²
		4300P95	
84A-6959	P95	X	A B C J M N O P

1. Protection

P95-Particulate Filter (95% filter efficiency level) effective against all particulate aerosols; time use restrictions may apply.

2. Cautions and Limitations

A- Not for use in atmospheres containing less than 19.5 percent oxygen.
 B- Not for use in atmospheres immediately dangerous to life or health.
 C- Do not exceed maximum use concentrations established by regulatory standards.
 J- Failure to properly use and maintain this product could result in injury or death.
 M- All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations.
 N- Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configurations as specified by the manufacturer.
 O- Refer to user's instructions, and/or maintenance manuals for information on use and maintenance of these respirators.
 P- NIOSH does not evaluate respirators as surgical masks.

USE AGAINST:

- Oil and non-oil particulate aerosols.

DO NOT USE AGAINST:

- Concentrations of contaminants which are unknown or are immediately dangerous to life or health.
- Concentrations of particulates which exceed the maximum use concentration or 10 times the OSHA Permissible Exposure Limit, whichever is lower.
- Gases, vapors, asbestos, paint spray, sandblasting or particulate materials which generate harmful vapors.
- Oil-based mists for longer than one 8 hour work shift.

RESTRICTIONS

- This respirator does not supply oxygen and must not be used in atmospheres containing less than 19.5% oxygen.
- Do not remain in contaminated area if any physical distress occurs, for example breathing difficulty, dizziness or nausea.
- Leave contaminated area and replace respirator if it is damaged, distorted, a proper fit can not be obtained or breathing becomes difficult.
- Prior to each use, carefully inspect the entire respirator, including filter media and strap/weld attachment area for tears and damage. Staple or strap/weld attachment perforations do not affect respirator performance.
- Do not alter, modify, or abuse this respirator.
- Store unused respirators in box/bag in a clean, dry, non-contaminated area.
- Dispose of respirator according to your employer's policy and local regulations.
- Where oil mists are present**, either alone or in combination with solid particulates, do not use for longer than one eight-hour work shift. Dispose no later than eight hours after first use. Do not exceed 10X PEL (see Do Not Use Against, item 2) and 200 mg loading per respirator. It is the user's responsibility to know the PEL and concentration of the contaminant, the maximum work rate of the employee, and any other work sitespecific information to calculate the loading of the filter. Where non-oil particulates **only** are present, dispose no later than 30 days after first use.
- Use respirator before the "use by" expiration date printed on box/bag.
- If used for welding, wear appropriate eye and face protection.

DISTRIBUTED BY:



Printed on Recycled Paper