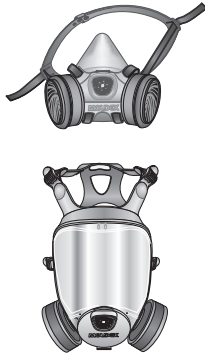
#0074 – Gasket Kit

Moldex® 7000 & 9000 Series Replacement Parts

The Moldex® 7000 & 9000 Series Respirator Gasket Kit includes 20 gaskets. Replace gaskets as needed.

9000 SERIES CARTRIDGE RETAINER

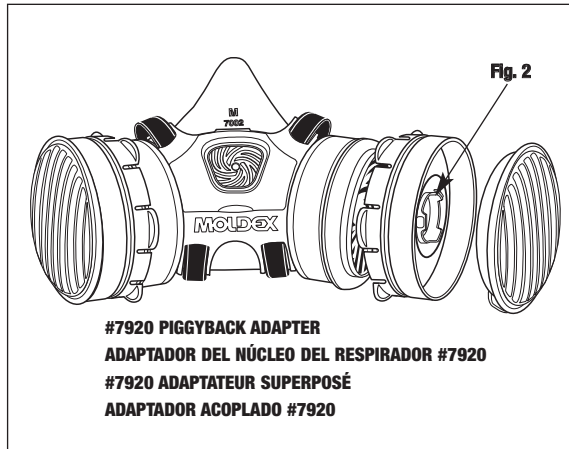
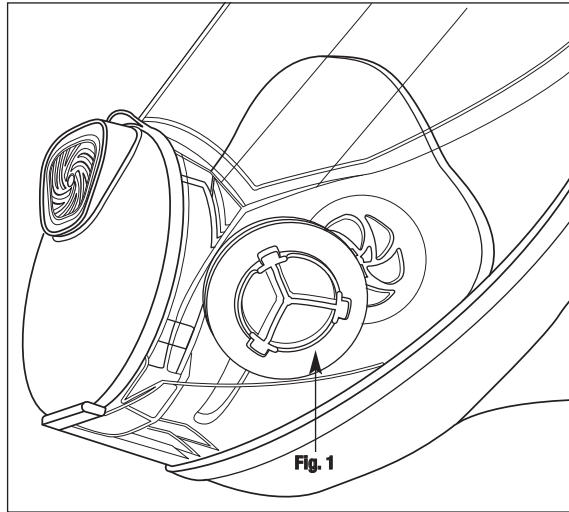
- Pull off old gasket over the three bayonet prongs on cartridge retainer and discard. (Fig. 1)
- Install new gasket by pushing gasket over the three bayonet prongs on cartridge retainer and ensure it is seated properly. Note: the 7000 Series facepiece does not have a gasket on its cartridge retainers.

#7920 PIGGYBACK ADAPTER


- Pull off old gasket over the three bayonet prongs on piggyback adapter and discard. (Fig. 2)
- Install new gasket by pushing gasket over the three bayonet prongs on piggyback adapter and ensure it is seated properly.

LIMITED WARRANTY

IMPORTANT NOTICE TO PURCHASER: This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, express or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Moldex will be met by products sold. **Exclusive Remedies:** damages for the breach of this limited warranty are limited to the replacement of such quantity of Moldex products proved to be defectively manufactured. Except as provided above, Moldex shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

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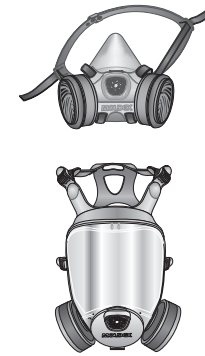CANADIAN CUSTOMER SERVICE
Tel: +1 (800) 421-0668, Ext. 517 Fax: +1 (310) 837-9563

BRAZIL – MOLDEX-METRIC LTDA
Av Ipiranga, 104, 22º Andar Sala 01
Sao Paulo, SP Brazil
Tel/Fax: +55 (11) 26 59 21 08
moldexbrazil@moldex.com

AUSTRALIA – MOLDEX-METRIC, INC.
Unit 30/14-16 Stanton Road, Seven Hills, NSW 2147
Tel: +61 (2) 9838 7588 Fax: +61 (2) 9838 7544
Email: moldex@bigpond.net.au www.moldex.com

NEW ZEALAND – MOLDEX-METRIC, INC.
3 Spence Road, Henderson, Auckland, NZ
Tel: +64 (9) 837 8247 Fax: +64 (9) 837 8248
Email: trevorc@moldex.com www.moldex.com

MOLDEX TECHNICAL SERVICE DEPARTMENT:
+1 (800) 421-0668 or +1 (310) 837-6500, ext. 512/550

Printed on Recycled Paper

Moldex and Ideas that wear well are registered trademarks. U.S. Patents Pending. The PVC-Free logo is a trademark of Moldex-Metric, Inc. 9000-024 REV C 12/10

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É.U. Brevets en attente. Le logo PVC-Free est une marque de commerce de Moldex-Metric, Inc.
A logomarca PVC-Free é uma marca registrada da Moldex-Metric, Inc.